






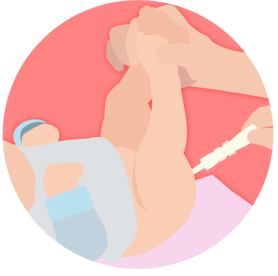

# BLUE FISH

---

## P E D I A T R I C S

Sometimes baby's full of gas. Skip the drops and use Windi to instantly free stubborn gas stuck in baby's pipes. It's completely safe, doctor-invented and the only natural instant solution for gas relief.

The Windi is a single-use catheter that helps babies get rid of excess gas. Designed to be safe and effective, The Windi cannot harm your baby if used as instructed.

<p><b>Step 1 of 5</b> Massage belly gently downward towards rectum. Massage one side of the belly at a time. Repeat three times on each side.</p>		<p><b>Step 2 of 5</b> Lubricate Windi tip. We suggest coconut or baby oil, but any lubricant will do.</p>	
<p><b>Step 3 of 5</b> Lift legs and have a diaper ready... trust us, it could get messy.</p>		<p><b>Step 4 of 5</b> Gently insert Windi and listen for the gas to pass (it will sound like a whistle) within a few seconds.*</p>	
<p><b>Step 5 of 5</b> Discard Windi.</p>		<p><b>Warning:</b> Never leave the catheter in the rectum of the child! Always discard The Windi after you have finished the treatment. Reuse of the same catheter may pose a risk of infection. Never use The Windi treatment more than three times per 24 hour period on your child.</p>	

\*It should only take a few seconds after insertion to get the desired effect. If it does not work immediately, repeat Steps 1-4 and try again. If you need to repeat several times, the same catheter can be used again.

**Manufactured by:** DiProServa Medical AB / Adelgatan 21, 211 22 Malmö, Sweden  
**Distributed by:** FridaBaby, LLC / [www.fridababy.com](http://www.fridababy.com)